



Expert information

durimplant

Manufacturer

lege artis Pharma GmbH + Co. KG
P. O. Box 60, D-72132 Dettenhausen
Breitwasenring 1, D-72135 Dettenhausen
Telephone +49 (0) 71 57 / 56 45 - 0
Fax +49 (0) 71 57 / 56 45 50
E-Mail: info@legeartis.de

Name of product

durimplant

Implant care gel

Prolonged protection of dental implants with effective gingival care

Ingredients

PARAFFINUM LIQUIDUM, HYDROXYPROPYL METHYLCELLULOSE, POLYETHYLENE, SILICA, ALLANTOIN, SALVIA LAVANDULIFOLIA, MENTHA PIPERITA, THYMOL, AQUA, CHLORHEXIDINE DIGLUCONATE, CHLORHEXIDINE DIACETATE, ASPARTAME, LINALOOL, LIMONEN.

Contains a source of phenylalanine.

Uses and Properties

durimplant is used to care for the gums and oral mucosa and to prevent inflammation in the vicinity of dental implants (e.g. peri-implantitis).

The viscous, stable gel adheres very well to the gingiva and oral mucosa. After being applied to the tissue – for example, around new dental implants – it forms a thin, soft protective layer containing sage and allantoin. *durimplant* is not easily washed off the tissue as it is water-repellent. It cares for gingiva and oral mucosa and prevents inflammation.

durimplant can also be used for all other forms of mild inflammation, irritation and pressure points in the mouth and throat for active care of the gums and oral mucosa in order to aid wound healing and tissue regeneration. As a long lasting film it serves additionally for protecting against chemical and mechanical irritants.

Application

In the dental practice:

Apply a thin layer of the required quantity of the viscous gel to the slightly moist gingiva around an implant or to the area to be treated, using a suitable instrument (e.g. single-tufted brushes from TePe: Compact Tuft, PZN^{*} 2745224 or Interspace x-soft, PZN^{*} 8864007). The gel can also be rubbed more deeply into the gingiva / oral mucosa, if necessary.

* PZN = German product code: Central Pharma Number

At home and when travelling:

Place a pea-sized quantity of gel onto a clean finger-tip, a cotton bud or some other suitable instrument (e.g. single-tufted TePe brushes: Compact Tuft, PZN* 2745224 or Interspace x-soft, PZN* 8864007), then apply to the moist gum area around the implant or to the area to be treated. The surface can then be smoothed and shaped gently with a finger moistened with saliva or the gel can be rubbed into the gums / oral mucosa.

For gingival care it should be applied once a week until once a day after the last cleaning of the teeth.

As required *durimplant* can be applied several times a day, in consultation with your dental practice.

Shelf life

The product should not be used beyond the date of minimum durability.

Dosage form and pack sizes

10 ml gel per tube

PZN* 4999590

Date of revision

August 2010

Further information available at www.durimplant.com

0026207/Stückzahl0810/Bestelldatum