



Expert information

LEGASED natur

Manufacturer

CE

lege artis Pharma GmbH + Co KG
P. O. Box 60, D-72132 Dettenhausen
Breitwasenring 1, D-72135 Dettenhausen
Telephone +49 (0) 71 57 / 56 45 - 0
Fax +49 (0) 71 57 / 56 45 50
E-Mail: info@legeartis.de
www.legeartis.de

Name

LEGASED natur
Oral wound dressing

Composition

Natural resins (myrrh, benzoin, colophony), ethanol, modified cellulose and macrogol

Indications

Viscous, film-forming fluid applied as a protective coating in cases of mild inflammation in the oral cavity, sore ulcer and aphthous stomatitis. Offers protection against mechanical pressure and chemical irritation, thus relieving existing pain.

Contra-indications

Known hypersensitivity to natural resins, particularly colophony, or to any of the other ingredients. Not for large, open wounds (e. g. Alveoli).

Side effects

When used correctly, side effects are not known.

Warnings

Contains approx. 23 % alcohol.

In case of eye contact, rinse eyes with plenty of water. Call a physician if necessary.

Remove any spilled or splashed solution as quickly as possible with alcohol or another suitable solvent (attention: do not use water or water-based cleaning agents, as this leads to immediate precipitation), as the natural resins contained in the solution will otherwise become sticky and can leave stains behind.

Mode of application

At the dental surgery:

Dab dry the area to be treated and keep as dry as possible to ensure the fluid adheres well. Then apply the solution, using a suitable instrument (e.g. Apply-Tip), to the dry mucous membrane (do not rub in).

If necessary, the solution can also be filled into a disposable syringe and applied to specific areas using a blunt and not too narrow cannula.

At home and away on holiday:

If possible, dab dry the area to be treated to enable the protective coating to adhere better (e.g. with a paper tissue). Then quickly apply a generous amount of the solution using a cotton-wool bud or, if necessary, clean fingertip (do not rub in). To speed up the film-forming process, the mouth may be briefly rinsed out with water. The solution can be applied several times daily, particularly after meals and before going to bed.

Further notices

The antibacterial barrier effect of the myrrh resin aids healing and tissue regeneration, and helps to relieve pain as a result.

Shelf life

LEGASED natur should not be used after the expiry date.
Once the container has been opened, the solution must be used within 6 months.

Storage and preservation

Always store LEGASED natur tightly closed.

Administrative form and package sizes

5 ml solution
20 ml solution (practice pack)

Date of revision

January 2010

We hope you get better soon!

Symbol for "Expiry date"



Symbol for "Batch number"



Symbol for "Pay attention to the directions for use"



0026329/Stückzahl0110/Bestelldatum