



Expert information

## **ORBAT sensitive**

### **Manufacturer**

CE

**lege artis** Pharma GmbH + Co. KG  
P. O. Box 60, D-72132 Dettenhausen  
Breitwasenring 1, D-72135 Dettenhausen  
Telephone +49 (0) 71 57 / 56 45 - 0  
Fax +49 (0) 71 57 / 56 45 50  
E-Mail: info@legeartis.de  
www.legeartis.de

### **Name**

ORBAT sensitive  
Liquid for gingival retraction

### **Composition**

Aluminium sulfate solution 25%

### **Indications**

For retraction of the gingiva.  
For haemostasis of gingival bleeding and for stopping the exudation before impression taking.  
For tissue management during direct and indirect restorations.

### **Precautions for use**

Do not use on patients with allergy to aluminium compounds.  
Do not use with impregnated retraction cords.  
Aluminium sulfate can lead to slight desquamation of the mucosal epithelium.  
In case of eye contact, rinse eyes with plenty of water. Call a physician if necessary.

### **Mode of application**

#### Retraction of the gingiva

For gingival retraction before impression taking, soak a non-impregnated retraction cord (ring) with ORBAT sensitive. Remove the cord after approx. 1 - 3 minutes (or even longer) and gently rinse if necessary.

#### Exudation and gingival bleeding

Swab away the exudate or blood and apply ORBAT sensitive with soaked cotton pellet or another suitable applicator under light pressure.

#### Tissue management during direct and indirect restorations

If heavier capillary bleeding needs to be handled, ORBAT forte should be used for gingival retraction and to stop gingival bleeding.

Otherwise, for gingival retraction and to stop exudation, use a suitable applicator to rub in ORBAT sensitive directly or lay a soaked retraction cord (ring) into the sulcus.

Before cementation or adhesive attachment of the restoration, care must be taken that any gingival bleeding and exudation are stopped and excess aluminium sulfate has been thoroughly removed.

### **Further notices**

Aluminium sulfate has a good astringent effect. It causes local haemostasis by contracting the upper layer of the gingiva.

ORBAT sensitive can also be applied from the dropper bottle on the applicator or to suitable retraction cords directly. However, it is better to add a few drops from the bottle in a dappen dish or something similar for wetting the cords or to use an applicator to take the solution from there.

Keep out of reach of children.  
Only for professional dental use.

### **Shelf life**

ORBAT sensitive should not be used after the expiry date.

### **Storage instructions**

None

### **Presentation form and package size(s)**

20 ml solution in PE dropper bottle

### **Date of revision**

July 2010

Symbol of "Expiry date"



Symbol of "Batch number"



Symbol of "Pay attention to the directions for use"



0026327/Stückzahl0710/Bestelldatum