



Expert information

LEGASED natur

Manufacturer

lege artis Pharma GmbH + Co. KG
P. O. Box 60, D-72132 Dettenhausen
Breitwasenring 1, D-72135 Dettenhausen
Telephone +49 (0) 71 57 / 56 45 - 0
Fax +49 (0) 71 57 / 56 45 50
E-Mail: info@legeartis.de
www.legeartis.de

CE

Name

LEGASED natur
Oral wound dressing

Composition

Natural resins (myrrh, benzoin, colophony), ethanol, modified cellulose and macrogol

Indications

Viscous, film-forming fluid applied as a protective coating in cases of mild inflammation in the oral cavity, sore ulcer and aphthous stomatitis. Protects against mechanical pressure and chemical irritation and thereby relieves existing pain.

Contra indications / Side effects

Known hypersensitivity to natural resins, particularly colophony, or to any of the other ingredients.

Notices and precautions

Contains approx. 23 % alcohol.

Not for large, open wounds (e.g. Alveoli).

Not to use in children under 12 years of age.

There are no or limited amount of data from the use of myrrh in pregnant or breast-feeding women. As a precautionary measure, it is preferable to avoid the use of LEGASED natur during pregnancy and breast-feeding.

If the symptoms worsen during the use of LEGASED natur or if there is no improvement within a week, a doctor or dentist should be consulted.

After swallowing, rinse mouth. If you feel unwell, seek medical advice / attention.

In case of skin contact, wash gently with plenty of water and soap. If skin irritation or rash occurs, seek medical advice/attention.

If eye contact occurs, rinse gently with water for a few minutes, remove any existing contact lenses if possible. Continue rinsing. Seek medical advice / medical attention immediately.

Remove any spilled or splashed solution as quickly as possible with alcohol or another suitable solvent (attention: do not use water or water-based cleaning agents, this leads to immediate precipitation), otherwise the natural resins contained in the solution become sticky and can leave stains.

The safety data sheet can be downloaded from www.legeartis.de or requested by E-Mail from sicherheitsdatenblaetter@legeartis.de.

Mode of application

At the dental surgery:

Pat dry the area to be treated and keep it as dry as possible to ensure better adhesion of the fluid. Then apply the solution with a suitable instrument (e.g. Apply-Tip) to the dry mucous membrane (do not rub in) and briefly moisten the application site for better film formation.

If necessary, the solution can also be drawn from the bottle into a disposable syringe and applied with a blunt and not too narrow cannula.

