(DE) Fachinformation
LEGASED natur

Hersteller

lege artis Pharma GmbH + Co. KG
Postfach 60, D-72132 Dettenhausen
Breitwasenring 1, D-72135 Dettenhausen
Telefon +49 (0) 71 57 / 56 45 - 0
Fax +49 (0) 71 57 / 56 45 50
E-Mail: info@legeartis.de
www.legeartis.de

Bezeichnung

LEGASED natur
Mund-Wund-Pflaster

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Indications

Viscous, film-forming fluid applied as a protective coating in cases of mild inflammation in the oral cavity, sore ulcer and aphthous stomatitis. Protects against mechanical pressure and chemical irritation and thereby relieves existing pain.

Contra indications

Known hypersensitivity to natural resins, particularly colophony, or to any of the other ingredients.

Side effects

Known hypersensitivity to natural resins, particularly colophony, or to any of the other ingredients. In the event of serious incidents in connection with the product, please contact the manufacturer and the responsible authority immediately.

Notices and precautions

Contains approx. 23 % alcohol.

Not for large, open wounds (e.g. Alveoli).

Not to use in children under 12 years of age.

There are no or limited amount of data from the use of myrrh in pregnant or breast-feeding women. As a precautionary measure, it is preferable to avoid the use of LEGASED natur during pregnancy and breast-feeding.

If the symptoms worsen during the use of LEGASED natur or if there is no improvement within a week, a doctor or dentist should be consulted.

After swallowing, rinse mouth. If you feel unwell, seek medical advice / attention.

In case of skin contact, wash gently with plenty of water and soap. If skin irritation or rash occurs, seek medical advice/attention.

If eye contact occurs, rinse gently with water for a few minutes, remove any existing contact lenses if possible. Continue rinsing. Seek medical advice / medical attention immediately.

Remove any spilled or splashed solution as quickly as possible with alcohol or another suitable solvent (attention: do not use water or water-based cleaning agents, this leads to immediate precipitation), otherwise the natural resins contained in the solution become sticky and can leave stains.

The safety data sheet can be downloaded from www.legeartis.de or requested by E-Mail from sicherheitsdatenblaetter@legeartis.de.

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