(DE) Fachinformation  
**LEGASED natur**

#### **Hersteller**

**lege artis** Pharma GmbH + Co. KG  
Postfach 60, D-72132 Dettenhausen  
Breitwasenring 1, D-72135 Dettenhausen  
Telefon +49 (0) 71 57 / 56 45 - 0  
Fax +49 (0) 71 57 / 56 45 50  
E-Mail: [info@legeartis.de](mailto:info@legeartis.de)  
[www.legeartis.de](http://www.legeartis.de)

#### **Bezeichnung**

LEGASED natur  
Mund-Wund-Pflaster

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**Indications**

Viscous, film-forming fluid applied as a protective coating in cases of mild inflammation in the oral cavity, sore ulcer and aphthous stomatitis. Protects against mechanical pressure and chemical irritation and thereby relieves existing pain.

**Contra indications**

Known hypersensitivity to natural resins, particularly colophony, or to any of the other ingredients.

**Side effects**

Known hypersensitivity to natural resins, particularly colophony, or to any of the other ingredients. In the event of serious incidents in connection with the product, please contact the manufacturer and the responsible authority immediately.

**Notices and precautions**

Contains approx. 23 % alcohol.

Not for large, open wounds (e.g. Alveoli).

Not to use in children under 12 years of age.

There are no or limited amount of data from the use of myrrh in pregnant or breast-feeding women. As a precautionary measure, it is preferable to avoid the use of LEGASED natur during pregnancy and breast-feeding.

If the symptoms worsen during the use of LEGASED natur or if there is no improvement within a week, a doctor or dentist should be consulted.

After swallowing, rinse mouth. If you feel unwell, seek medical advice / attention.

In case of skin contact, wash gently with plenty of water and soap. If skin irritation or rash occurs, seek medical advice/attention.

If eye contact occurs, rinse gently with water for a few minutes, remove any existing contact lenses if possible. Continue rinsing. Seek medical advice / medical attention immediately.

Remove any spilled or splashed solution as quickly as possible with alcohol or another suitable solvent (attention: do not use water or water-based cleaning agents, this leads to immediate precipitation), otherwise the natural resins contained in the solution become sticky and can leave stains.

The safety data sheet can be downloaded from [www.legeartis.de](http://www.legeartis.de) or requested by E-Mail from [sicherheitsdatenblaetter@legeartis.de](mailto:sicherheitsdatenblaetter@legeartis.de).

## Mode of application

### At the dental surgery:

Pat dry the area to be treated and keep it as dry as possible to ensure better adhesion of the fluid. Then apply the solution with a suitable instrument (e.g. Apply-Tip) to the dry mucous membrane (do not rub in) and briefly moisten the application site for better film formation.

If necessary, the solution can also be drawn from the bottle into a disposable syringe and applied with a blunt and not too narrow cannula.

### At home and outside home :

Pat dry the area to be treated as dry as possible to enable better adhesion of the protective coating (e.g. with a paper tissue). Then apply the solution quickly with a cotton swap or, if necessary, a clean fingertip (do not rub in). The mouth can be briefly rinsed out with water to more rapid film formation. The application can take place several times a day, especially after meals and before going to bed.

## Further notices

The antibacterial barrier effect of the myrrh resin supports wound healing and tissue regeneration in a positive way and thus contributes to pain relief.

Keep out of reach of children.

Do not pour back residues of the solution into the bottle.

## Shelf life

LEGASED natur should not be used after the expiry date.

Once the container has been opened, the solution must be used within 6 months.

## Storage and preservation

Close the bottle of LEGASED natur immediately after use and keep tightly closed.

## Dosage form and package sizes

5 ml solution                      Item number 0032329

20 ml solution (practice pack)      Item number 0032330

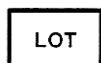
## Date of revision

2023-08

Symbol of "Expiry date"

Symbol of "Batch number"

Symbol of "Pay attention to the directions for use"

Symbol of "Manufacturer"

Symbol of "Catalogue number"  
Shows Item number of the manufacturer

Symbol of "Medical Device"

Symbol of "Unique device identification"

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