Hersteller

lege artis Pharma GmbH + Co. KG
Postfach 60, D-72132 Dettenhausen
Breitwasenring 1, D-72135 Dettenhausen
Telefon +49 (0) 71 57 / 56 45 - 0
Fax +49 (0) 71 57 / 56 45 50
E-Mail: info@legeartis.de
www.legeartis.de

Bezeichnung

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Name

TECTOR
Calcium hydroxide suspension for pulp protection
For dental use only

Composition

Calcium hydroxide, Canada balsam and Ethylmethylketone

Indications

TECTOR serves to protect pulps after grinding down and preparing of vital teeth, to avoid and eliminate pain during grinding, prior to the cementing of crowns and bridges and to dab shallow cavities.

Side effects

In particular cases, hypersensitivity reactions (allergies) cannot be excluded. Known hypersensitivity to natural resins, particularly canada balsam, or to any of the other ingredients are possible.

Notices and precautions

Do not use if subsequently adhesive cementation (bonding) shall take place.

Do not use if pulp opening occurs during grinding.

When applying TECTOR ensure that TECTOR does not flow over the gingiva and on the oral mucosa (use rubber dam) since calcium hydroxide reacts highly alkaline with moisture. By the alkalinity a burning pain can momentarily arise. If TECTOR has flown on the oral mucosa, it must be immediately removed with a cotton wool pellet soaked with water.

The liquid is easily inflammable and should not come into contact with open flames.

After swallowing, rinse mouth. Get immediate medical advice / attention.

In case of skin contact, remove mechanically (e.g. dab away using cotton or cellulose material) then thoroughly wash the affected skin with a mild cleansing agent and water. If skin irritation or rash occurs, seek medical advice / attention.

If eye contact occurs, rinse gently with water for a few minutes, remove any existing contact lenses if possible. Continue rinsing. Seek medical advice / medical attention immediately.

Avoid product contact with clothing.

The safety data sheet can be downloaded from www.legeartis.de or requested by E-Mail from sicherheitsdatenblaetter@legeartis.de.

The summary of safety and clinical performance according to Regulation (EU) 2017/45, Article 32 can be found at www.legeartis.de

Mode of application

Prior to the application the areas to be treated must be clean, dry, and free from fat. Shake TECTOR well before use until the calcium hydroxide particles are evenly distributed in the liquid.

Apply the suspension with a suitable applicator (e.g. Apply-Tip, Microbrush) and with a suitable technique (protection of the gingiva and oral mucosa by using a rubber dam) dab three times if possible so that a sufficiently thick layer results. Allow to dry on after each dabbing. Do not dry with the air-syringe or the calcium hydroxide particles will be unevenly distributed.

The liquid evaporates very quickly and an extremely thin natural resin layer remains in which the calcium hydroxide particles are embedded.

TECTOR shall not be applied up to the cavity margin. If necessary, the projecting layer can be easily removed with a spatula.

After drying of the TECTOR layer the temporary protective capping, the definite crown or the filling can be applied.

If during the removal of a temporary crown or a filling the TECTOR layer is damaged, the area concerned can be cleaned with a suitable solvent and dried and TECTOR can be applied again.

In exceptional cases the shaking up of TECTOR may be reduced. We recommend in this case, to put TECTOR into an ultrasonic bath until the sediment will be completely dispersed.

Further notices

When grinding the teeth and preparing the cavities dental canaliculi are being incised and exposed which are open over their full length up to the pulp. A dentin wound results. These open dental canaliculi are very susceptible to noxae, toxins, and microorganisms. By the application of TECTOR these open dental canaliculi are closed.

The calcium hydroxide-containing TECTOR layer protects the pulp from penetration of residual acids when cementing crowns and when filling shallow cavities.

Because of the accurate fit of the crowns and bridges TECTOR must not be applied in too thick layers on ground stumps.

Do not apply TECTOR together with an adhesive cementation (bonding).

Do not apply TECTOR on composite teeth, composite veneer crowns and composite fillings.

The flowing off of TECTOR from the application area on the gingiva or the oral mucosa must be avoided since calcium hydroxide together with the humidity can lead to cauterization.

TECTOR shows an intensive signal-odour. For the patients' better acceptance it has been proved useful to inform them before the application of TECTOR that it will smell - for example - like nail polish remover.

Keep out of reach of children.

Do not pour back residues of the solution/suspension into the bottle.

Shelf life

TECTOR shall not be used after the expiry date.

Storage instructions

After use, the TECTOR container must be tightly closed or the liquid will evaporate quickly.

Dosage form and package size

10 ml suspension Item number 0032310

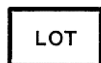
Date of revision

2023-02

Symbol of "Expiry date"

Symbol of "Batch number"

Symbol of "Pay attention to the directions for use"

Symbol of "Manufacturer"

Symbol of "Catalogue number"

Shows Item number of the manufacturer

Symbol of "Medical Device"

Symbol of "Unique device identification"

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