Hersteller

lege artis Pharma GmbH + Co. KG
Postfach 60, D-72132 Dettenhausen
Breitwasenring 1, D-72135 Dettenhausen
Telefon +49 (0) 71 57 / 56 45 - 0
Fax +49 (0) 71 57 / 56 45 50
E-Mail: info@legeartis.de
www.legeartis.de

Bezeichnung

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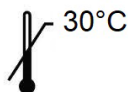
Name

ORBAT sensitive
Liquid for gingival retraction
For dental (dentist) use only

Composition

Aluminium sulfate solution 25%

Intended purpose

Due to its chemical and physical properties, ORBAT sensitive is used for gingival retraction prior to impression taking.

Indications

For retraction of the gingiva.
For tissue management during direct and indirect restorations.
For haemostasis of gingival bleeding and for stopping the exudation before impression taking.

Contra indications

Do not use on patients with allergy to aluminium compounds.

Side effects

Allergic reactions to aluminium compounds are possible.
Aluminium sulfate can lead to slight desquamation of the mucosal epithelium.

In the event of serious incidents in connection with the product, please contact the manufacturer and the responsible authority immediately.

Notices and precautions

Do not use with impregnated retraction cords.
If heavier capillary bleeding needs to be handled, ORBAT forte should be used for gingival retraction and to stop gingival bleeding.

Avoid swallowing. After swallowing, rinse mouth. Drink plenty of water in small sips (dilution effect). If you feel unwell, seek medical advice / attention.

In case of skin contact, wash gently with plenty of water and soap. If skin irritation occurs, seek medical advice / attention.

If eye contact occurs, rinse gently with water for a few minutes, remove any existing contact lenses if possible. Continue rinsing. Seek medical advice / medical attention immediately.

The safety data sheet can be downloaded from www.legeartis.de or requested by E-Mail from sicherheitsdatenblaetter@legeartis.de

Do not pour back residues of the solution into the bottle.

Keep out of reach of children.

Please also note the instructions under **Mode of application!**

Mode of application

Retraction of the gingiva

For gingival retraction before impression taking, soak a non-impregnated retraction cord (ring) with ORBAT sensitive. Remove the cord after approx. 1 - 3 minutes (or even longer) and gently rinse if necessary.

Tissue management during direct and indirect restorations

For gingival retraction and to stop exudation, lay a ORBAT sensitive soaked retraction cord (ring) into the sulcus or use a suitable applicator to rub in ORBAT sensitive directly under light pressure.

Before cementation or adhesive attachment of the restoration, care must be taken that any gingival bleeding

and exudation are stopped, and excess aluminium sulfate has been removed thoroughly with plenty of water spray.

Exudation and gingival bleeding

Swab away the exudate or blood and apply ORBAT sensitive with soaked cotton pellet or another suitable applicator under light pressure.

Further notices

Aluminium sulfate has a good astringent effect. It causes local haemostasis by contracting the upper layer of the gingiva.

ORBAT sensitive can be applied directly from the dropper bottle to the applicator or to suitable retraction cords. However, it is better to add a few drops from the bottle into a dappen dish or something similar in order to soak the cords there well or to take the solution with the applicator from there.

Shelf life

ORBAT sensitive should not be used after the expiry date.

Storage instructions

Close the bottle immediately after use.

Dosage form and package size

20 ml solution in PE dropper bottle Item number 0032327

Date of revision

2025-08

Symbol of "Expiry date"

Symbol of "Batch number"

Symbol of "Pay attention to the directions for use"

Symbol of "Manufacturer"

Symbol of "Catalogue number"
Shows Item number of the manufacturer

Symbol of "Medical Device"

Symbol of "Unique device identification"

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